



Kairax

Enhance 45




Official Online Training Now Available

Introduction

Kairax Enhance is a next-generation injectable treatment that combines High and Low Molecular Weight Hyaluronic Acid (H-HA and L-HA) with Polynucleotides (PN) for comprehensive skin rejuvenation.

This unique formula targets hydration, skin structure, and lifting at both surface and deeper layers. Ideal for achieving a youthful, radiant complexion, Kairax Enhance leaves skin smooth, firm, and revitalized.

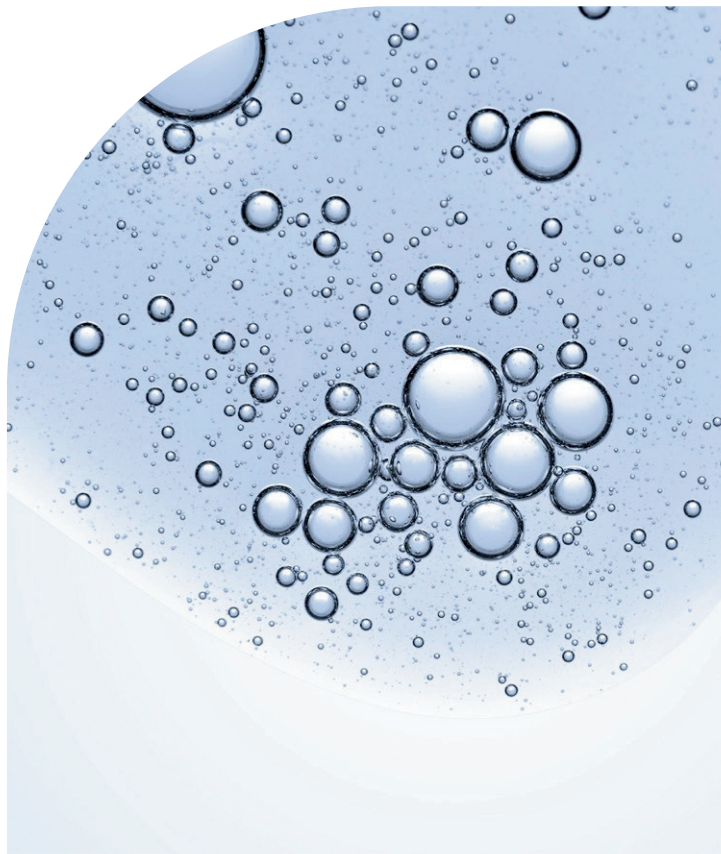



The image features a product box for Kairax Enhance 45, which is tilted diagonally. The box has a color gradient from orange at the top to pink at the bottom. It is set against a background of water ripples. A faint, large, grey watermark of the product name is visible in the background. At the bottom of the box, there is a geometric pattern of overlapping triangles in shades of pink and purple.

Kairax Enhance 45

45mg (H-HA) + 45mg (L-HA) / 2ml



High molecule HA (H-HA) and Low molecule HA (L-HA) + PN

High Molecular Weight Hyaluronic Acid (H-HA):

Provides intense surface hydration,
locking in moisture to plump and smooth the skin instantly.


Low Molecular Weight Hyaluronic Acid (L-HA):

Penetrates deeper into the dermis, stimulating collagen production
and improving skin structure, elasticity, and overall appearance over time.

Polynucleotides (PN):

These DNA-based molecules enhance cellular repair and regeneration,
boosting collagen synthesis,
improving skin elasticity, and promoting healing at a deep, cellular level.

How Kairax Enhance Works

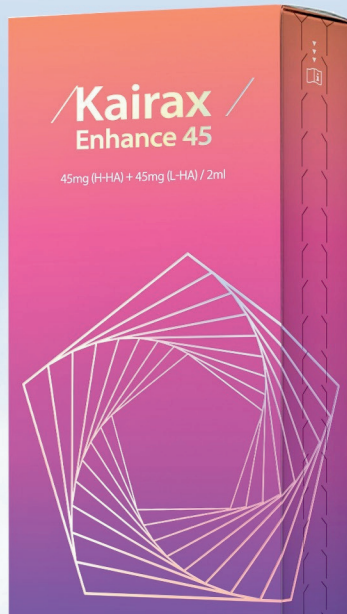

01 Instant Hydration & Surface Plumping

High Molecular Weight Hyaluronic Acid (H-HA) locks in moisture and plumps the skin's surface, reducing fine lines for an immediate smoothing effect.

02 Deep Skin Rejuvenation & Collagen Stimulation

Low Molecular Weight Hyaluronic Acid (L-HA) penetrates deeper into the dermis, stimulating collagen production and improving skin elasticity for long-term skin regeneration.

03 Cell Regeneration & Overall Skin Health


Polynucleotides (PN) promote cell renewal and enhance collagen synthesis, improving skin texture, firmness, and radiance for a youthful, revitalized appearance.


Advantages of Kairax Enhance

Superior hydration thanks to high concentration of HA.

Improved skin structure and lifting effect
with low and high molecule HA stability.

Enhanced skin regeneration with PN.

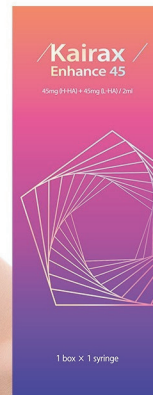

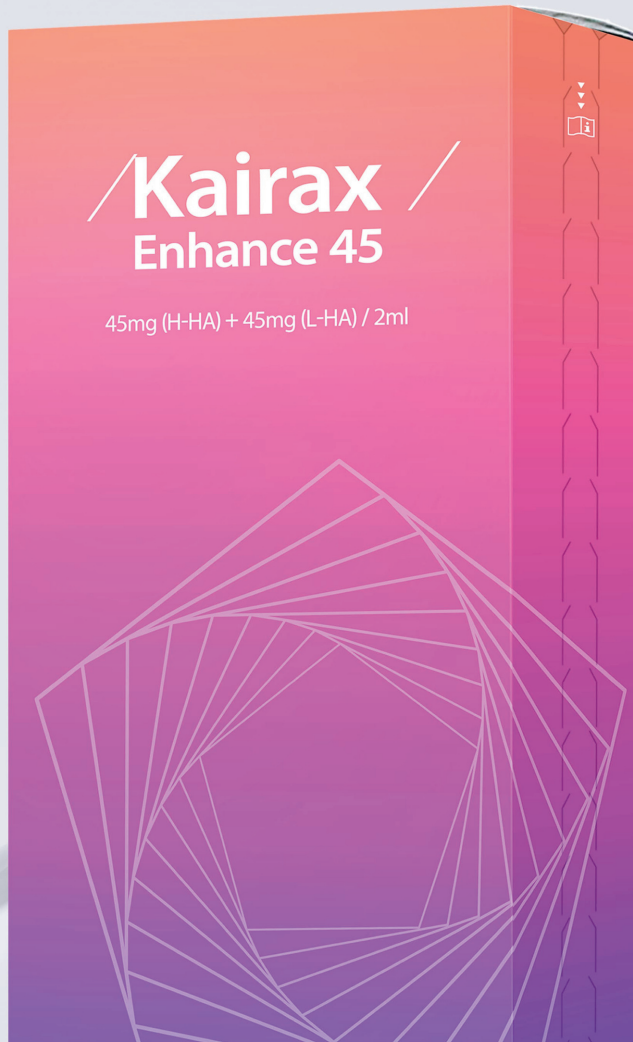


/Kairax / Enhance 45

Kairax Enhance is a next-generation injectable treatment that combines High and Low Molecular Weight Hyaluronic Acid with Polynucleotides (PN) for comprehensive skin rejuvenation.

This unique formula targets hydration, skin structure, and lifting at both surface and deeper layers. Ideal for achieving a youthful, radiant complexion, Kairax Enhance leaves skin smooth, firm, and revitalized.

Who needs Kairax Enhance?

CASE 1

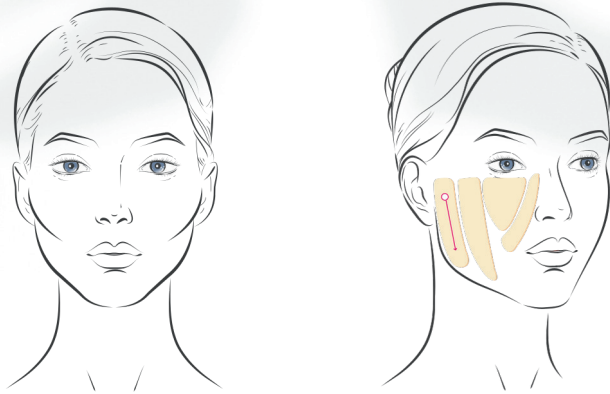

Key characteristics include

- Sunken appearance in the cheeks.
- Skin becomes thinner with reduced tissue density.
- Bones are more prominent, particularly around the temples and beneath the cheekbones.


Procedure

For Case 1 it's directed that:

- **Kairax Enhance** should be injected into the superficial fat compartment of the preauricular area.
- A retrograde injection technique should be used with a **27G x 50mm cannula**.
- **1ml of product** should be used on each side.

Who needs Kairax Enhance?

CASE 2

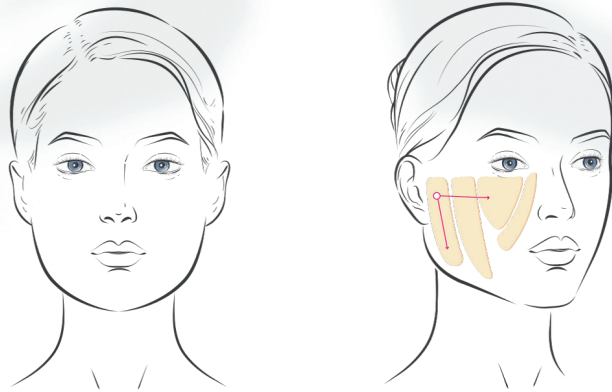

Key characteristics include

- Skin has a generally thicker and firmer feel.
- Volume tends to accumulate in the lower face.
- Loss of support in skin and tissue leads to more noticeable sagging and drooping.

Procedure

For Case 2 it's directed that:

- **Kairax Enhance** should be injected into superficial fat compartment of the preauricular area and zygomatic arch.
- A retrograde injection technique should be used with a **30G x 35mm cannula**.
- **0.5ml of product** should be injected into the preauricular area on each side, and 0.5ml should be administered to the zygomatic arch on each side.
- The entry point should be **3cm from the tragus**.

Number of treatments:



Significant results after only 1 month. (2 injections)

Ingredients:

High Molecular Weight Hyaluronic Acid (H-HA): 45mg

Low Molecular Weight Hyaluronic Acid (L-HA): 45mg

PN: 4mg

2ml prefilled syringe





Polynucleotides & Hyaluronic Acid Hybrid Aesthetic Points

- **First line - Start with the highest lateral point (2cm away from the orbital rim).**
 - > Follow the Zygomatic Bone - 1-1.5cm between each point (0.1ml each point)
- **Second Line - Start first point (2-3cm away from the mid tragus)**
 - > Follow down towards the commissure of the lips
- **Point 7** - 1cm down from the lip commissure, then 2cm outwards (towards the mandible corner)
- **Point 8** - 2cm from the nasal alar crease
- **Point 9** - On the angle of the mandible
- **Point 10** - 1cm down from the lip commissure, then 1cm inwards towards the mentalis crease

2ml per syringe - 10 points per side - 0.1ml per point



Micro Cannula 5 Point Technique

- **Point 1** - Over the zygomatic bone (0.2ml per fan)
- **Point 2** - Over the mandible angle region (0.2ml per fan)
- **Point 3** - Just over the mandible bone to be placed in the buccal region (0.2ml per fan)
- **Point 4** - To be spread over the mid malar region (0.2ml per fan)
- **Point 5** - To be spread over the mentalis region (0.2ml per fan)

30g x 25mm Micro Cannula or 27g x 50mm (for slightly deeper placement).

/Kairax /

Enhance 45



Exclusive at LPG Clinics Wholesale!

Email: info@lpgclinicswholesale.com