Feel the Bounce, See the Glow

AURORA


RH COLLAGEN

AURORA

recompinant Human Collagen 20 mg H-HA 25 mg + L-HA 25 mg

2.5ml x 1ea

RENAISSANCE BIO

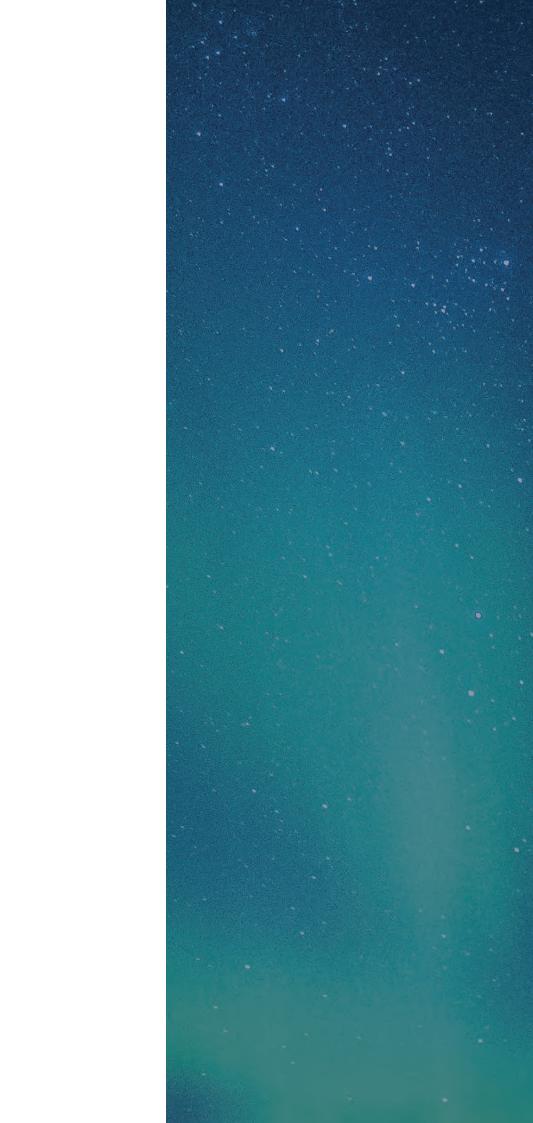

The colors of your skin's radiance

AURORA is a medical cosmetic brand inspired by one of nature's most mysterious and beautiful lights - the aurora.

Just as the aurora illuminates the darkness and heralds a new beginning, AURORA delivers regeneration and restoration to tired skin, bringing forth a luminous transformation



Our mission is to awaken the skin's innate power, creating a complexion that is healthy, smooth, and radiant.

Beyond skincare, AURORA aspires to add more beautiful colors to the shining moments of our customers' lives, offering not just treatment, but an experience that makes every day extraordinary.

## RH COLLAGEN



VOLUME 2.5 ml x 1syr / box

RH COLLAGEN 10 mg
TYPE I

RH COLLAGEN 10 mg
TYPE III

NON CROSS-LINKED 25 mg HA (HIGH MOLECULAR)


NON CROSS-LINKED 25 mg HA (LOW MOLECULAR)




#### Collagen and Aging

#### A Young Tissue Without Aging


Fibroblasts in the dermis are flattened and spread out, closely interacting with numerous intact collagen fibers.

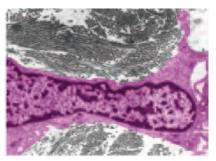


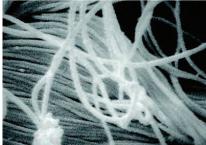

#### Aged Tissue

Fibroblasts in aged dermis appear collapsed with minimal cytoplasm.
Collagen fibers are damaged, fragmented, and lack close contact with fibroblasts.

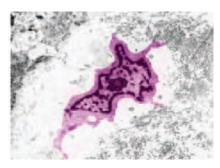

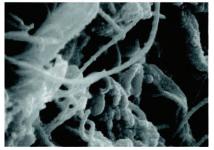

Collagen production decreases by 1% annually from the mid-20s. Improvement is possible through the direct supply of collagen from external sources.









Collagen makes up more than 70% of the skin and is the most critical component for determining skin aging and elasticity.




#### Whart is RH Collagen?





Recombinant Human Collagen (RHC) is a bioengineered form of collagen that mimics natural human collagen. (Identical to Human collagen) Produced through a controlled, ethical process. Free from animal-derived components, reducing the risk of allergic reactions. Ensures high purity and consistency, making it ideal for skincare applications.

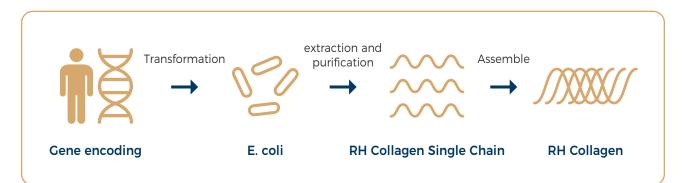

#### Benefits of RH Collagen



## Identical to Human Collagen

Safe, Biocompatible Seamlessly integrates into skin and tissue.



#### 100% Animal-Free

Ethical sourcing Cruelty-free Now religious restrictions



#### Reduced Allergenicity


Lower allergic reactions Suitable even for sensitive skin


## AURORA RH COLLAGEN

#### Benefits of RHC\* for the Skin

\*Recombinant Human collagen

RH Collagen can significantly improve the microcirculation of deep second-degree burned wounds, reduce the infiltration of inflammatory cells and promote the healing of burn wounds.

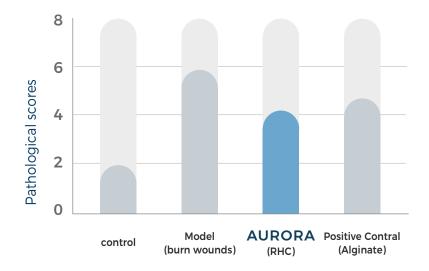

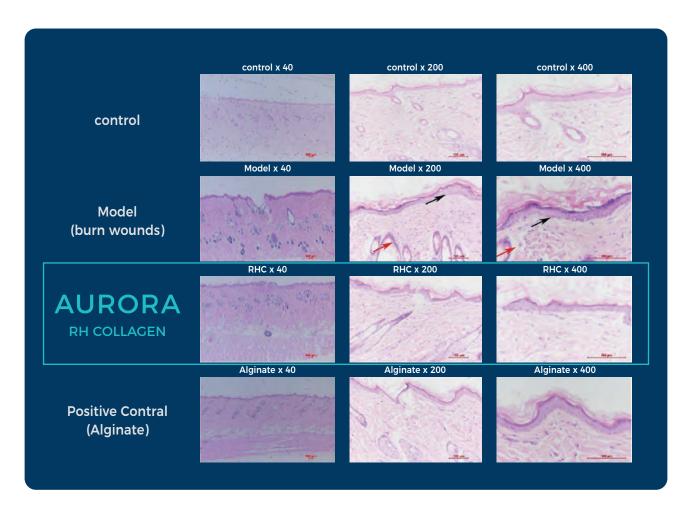



#### Why AURORA? ( ${ m I}$ ) - An Ideal Collagen Combination

#### • Collagen Type I "Structure and Pillar"

Type I acts as the skin's structural pillar, while Type III adds flexibility and elasticity—together forming a well-balanced skin architecture.

#### • Collagen Type Ⅲ "Elasticity and Resilience"

Type III first fills and repairs damaged areas, followed by Type I reinforcing structural integrity—leading to faster and stronger regeneration.


#### Type I + Type III

When both types are present, they enhance skin density, elasticity, and hydration evenly, resulting in a more youthful, healthy-looking complexion



#### Why AURORA? ( ${\mathbb I}$ ) - Dual HA Complex

AURORA RH COLLAGEN combines both high and low molecular weight hyaluronic acid (HA) to optimize skin vitality. High molecular weight HA helps restore volume and delivers a gentle lifting effect, while low molecular weight HA deeply hydrates the epidermis for lasting moisture. Together, they also stimulate collagen and elastin production, supporting skin rejuvenation and a healthier, more radiant complexion.

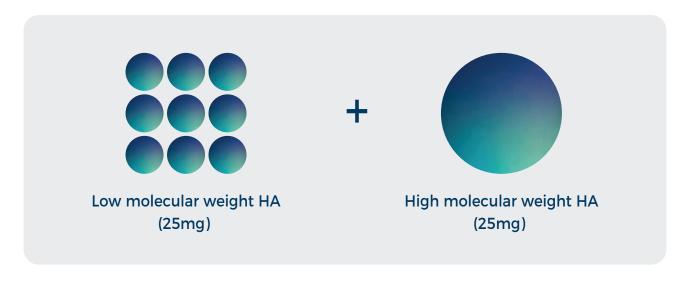


✓ Collagen Stimulation ✓ Elastin Production ✓ Moisturization

Hyaluronic Acid 50mg
(L-HA 25mg+ H-HA 25mg)

Moisture Recharge

#### **Protocol**

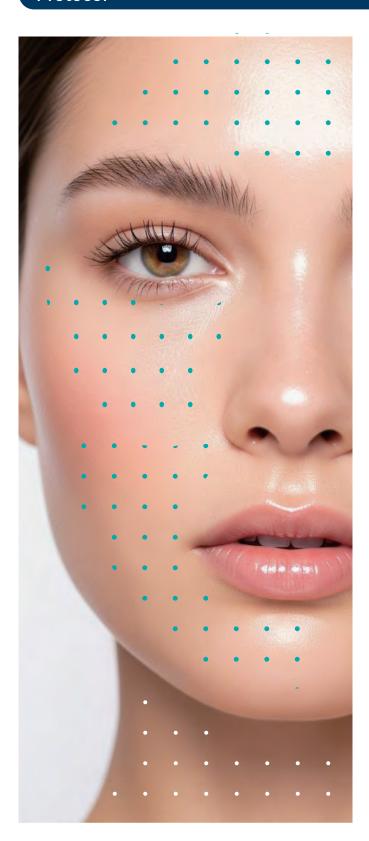

#### • Recommended Needle Gauge

The use of a 31G or 32G needle with a length of 4mm or 6mm is generally recommended.

#### • Injection Volume per Point

The general recommended usage is approximately 0.01-0.03 ml per point.

#### Injection Depth

Injection should be into the upper to mid dermis. In terms of depth, it is approximately 1.0~2.0 mm, which is a method of direct injection into the dermal layer beneath the epidermis.

#### Additional Sessions

The most stable and maintained results are expected with a total of **3 to 5 sessions**, performed at 2-3 week intervals.

For maintenance after the initial sessions, a booster session every quarter (3 months) is recommended.

#### **Precautions During and After the Procedure**

- 1. Before the procedure, always check the treatment area for any skin damage or inflammation.
- 2. For 24 hours after the procedure, avoid saunas, strenuous exercise, and alcohol consumption.
- Strictly adhere to the disinfection steps before and after the procedure to prevent infection.

#### Contraindications

- 1. Not permitted for: Pregnant women, breastfeeding women, or individuals with a keloid predisposition.
- Do not use on areas with: Active acne, dermatitis, or infectious skin dispases.
- 3. It is recommended to avoid treatment: Immediately after other skin procedures or within 1 week of laser treatments/peels.
- 4. Essential check prior to procedure: Confirm any allergies to topical anesthetics or the product's ingredients.

### Before & After



### **RH COLLAGEN**

**Product** Name

**AURORA RH COLLAGEN** 

Capacity

2.5 ml x 1syr / box

Ingredient

Recombinant Human Collagen 20 mg (Type I 10mg + Type III 10mg)

Hyaluronic Acid 50mg (L-HA 25mg+ H-HA 25mg)

Shelf-Life

36 months

RENAISSANCE BIO

LPG CLINICS WHOLESALE

www.lpgclinicswholesale.com

E. Info@lpgclinicswholesale.com



+44 7960874222